This instruction is for information only. Use before opening the container.  
Please read carefully and retain for future reference.

# MeniCare Plus

**EN MULTIPURPOSE SOLUTION**  
For all rigid gas permeable lenses

## INDICATIONS

- CLEANING
- RINSING after cleaning
- DISINFECTING : effective within 5 minutes
- SOAKING
- RINSING before insertion and LUBRICATING

## DIRECTIONS

- Wash your hands thoroughly before handling your contact lenses :
- Rub the lens carefully with several drops of MeniCare Plus solution during 20 seconds.
  - Place the lenses in the Right and Left lensholder and rinse them with MeniCare Plus.
  - Fill the lens case with MeniCare Plus and soak the lenses for a minimum of 5 minutes or overnight.
  - Before insertion, rinse the lenses with MeniCare Plus.

## WARNINGS

- Renew the solution in the lens case after each use and at least once a month in case of extended soaking periods due to lens wear interruption.
- Keep out of children's reach.
- Never use tap water or non-sterile water with contact lenses as microbial contamination can occur, which may lead to permanent eye damage.
- Consult your doctor or eye care professional before using any eye medication with contact lenses.

## PRECAUTIONS

- Once opened, use MeniCare Plus within 3 months and make sure that the bottle is tightly closed after each use.
- Do not touch the bottle top with fingers.
- Store at room temperature ( $\leq 25^{\circ}\text{C}$ ).
- Observe the expiry date printed on the box and the bottle.
- Not to initially use the product, if the safety ring is damaged.
- Do not swallow.
- Clean the lens case regularly and renew it every 3 months.
- In case of irritation, remove the lenses and consult your eye care practitioner.
- Store out of sunlight ☀

## CONTRA-INDICATIONS

- Do not use with soft contact lenses.
- Do not use in case of allergic background to one of ingredients.

## ADVERSE REACTIONS

If you experience persistent discomfort or irritation, remove your lenses, stop using the product and consult your eye care professional.

## CONTENTS

Polyhexamethylene Biguanide .....	0.0005 %
Poloxamer .....	0.5 %
Hypromellose .....	0.275 %

## PACKAGING

Menicare Plus is available in 250 ml, 125 ml and 50 ml bottles.

 **Fabricant / Fabricante / Fabricante / Produttore :**  
**Nicon Pharma SAS**  
Bd Sébastien Brant  
Parc d'Innovation - 67400 Illkirch Graffenstaden - France

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0483 © 2000,
Menicon Pharma SAS